

# Ecoflac<sup>®</sup> plus

## Practice suggestions: Pressure infusion

### 1 REMOVE SEALING FOIL



Remove one of the tamper-evident sealing foils on the Twincap<sup>®</sup> port. The surface below the foil is germ-free, it is not required to disinfect prior to use. Both ports are identical and can be used irrespectively for preparation or infusion.

### 2 INSERT SPIKE



While the container is standing, insert the infusion set spike into the open port at a straight (90°) angle, without rotation. Close the air vent of the infusion set (if applies).

### 3 EXPEL AIR FROM CONTAINER



Ensure the roller clamp of infusion set is open. Hold the Ecoflac<sup>®</sup> plus upright and press the container with both hands in order to expel all air via the line until the drip chamber is half filled. Close the roller clamp of infusion set while maintaining pressure on the container.

### 4 PRIME THE LINE



Turn the container upside down and open the roller clamp of infusion set to remove all air from the line. Close the roller clamp of infusion set.

### 5 INSERT IN PRESSURE CUFF



Place the container turned at a 90° angle into the pressure cuff (with the label on the side) and hang to the infusion stand. Pressure must be applied on the side of the container and not on the front. Build up desired pressure using pressure device.

### 6 START INFUSION



Connect the line to the patient access. Open the roller clamp of infusion set to start pressure infusion. It is normal to observe a residual at the end of a pressure infusion.

**IMPORTANT: Ecoflac<sup>®</sup> plus contains a certain amount of air. Elimination of all air before pressure infusion is mandatory.**