

# Prontosan® Wound Spray

Cleansing and moistening of superficial wounds and burns

B. Braun Wound Care. Right. From the start.



Wound Management

# Prontosan® Wound Spray



**HANDLING/APPLICATION TIP**

Rinse off residues of soaps, ointments, oils, or other wound cleansing substances thoroughly prior to the use of Prontosan® Wound Spray.

The **Prontosan® Wound Spray** supports rapid wound healing by effective cleansing and moistening of superficial wounds and burns.

**Application**

Thoroughly spray **Prontosan® Wound Spray** onto the entire wound and the wound site. With this, the risk for infection is reduced and optimal healing conditions are generated. Afterwards, cover the wound with a dressing.

Clotted or encrusted dressings can be moistened with **Prontosan® Wound Spray** in order to be easily removed and to avoid pain during removal.

**Advantages**

- › Prevents wound infection
- › Supports wound healing
- › Prevents scar formation
- › Prevents MDRO contamination
- › Prevents biofilm formation
- › Reduces pain
- › Painless application

**Indications**

For cleansing, irrigation and moistening of

- › Superficial acute and chronic wounds (e.g. skin lesions, bites, lacerations, abrasions, crush wounds, decubitus ulcers, ulcerus cruris, diabetic ulcers)
  - › Superficial burns
- Supports removal of encrusted bandages.

Prontosan® Wound Spray	Size	Pcs/Pack	Article number
Spray bottle	75 ml	20	400565
Composition: Purified Water, Betaine Surfactant, 0.1 % Polyaminopropyl Biguanide (Polihexanide)			



